




Pumpa

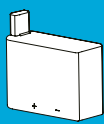


Baterija pumpe

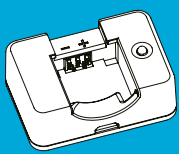

Punjač za bateriju pumpe

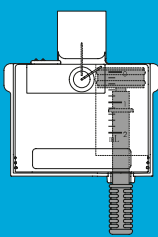


Spremnik s adapterom za punjenje

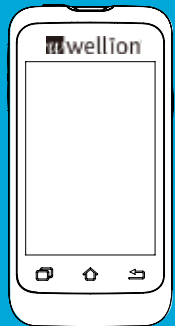


PDA – Prijenosni dijabetes asistent



PDA punjač

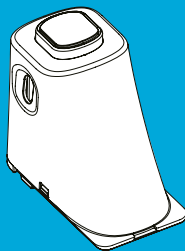

Aplikator

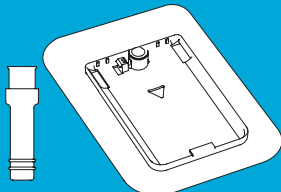

Infuzijski set (kanila+baza)

Sustav inzulinske pumpe

Kratki vodič

MicroTech Medical (Hangzhou) Co., Ltd.
No. 108 Liuze St., Cangqian, Yuhang District,
Hangzhou, 311121 Zhejiang, P.R.China



MED TRUST
THE MEDICAL SERVICES COMPANY

Service and distribution:
MED TRUST Handelsges.m.b.H.
Gewerbepark 10 • 7221 Marz, AUSTRIA
www.medtrust.at; www.wellion.at
Tel.: + 43 2626 64 190 • Fax: + 43 2626 64 190 - 77
E-Mail: office@medtrust.at

CE 0197


WELL19-37HR
20220510

1. Priprema komponenti pumpe


Ovaj odjeljak opisuje kako pripremiti komponente pumpe: baterije, spremnik i infuzijski set. Spremnike i infuzijske setove treba zamijeniti svaka tri dana.

1A Pripremite bateriju

1 Napunite bateriju pumpe

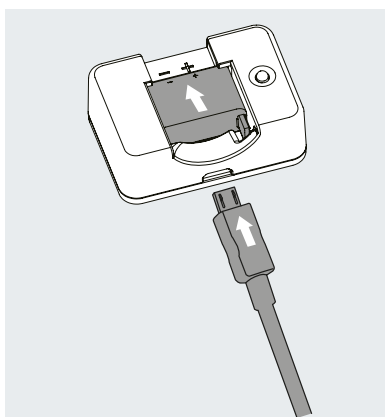


2 Napunite bateriju PDA

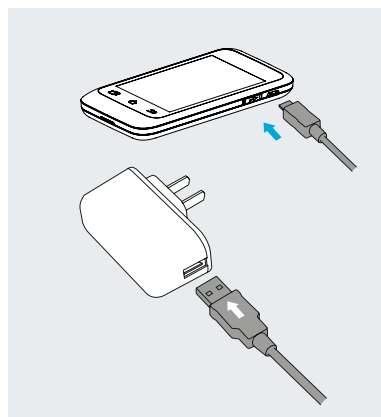


1B Pripremite novi spremnik


1 Izvadite spremnik iz ambalaže. Očistite bočicu i pričvrstite ju na adapter za punjenje kako biste ga napunili.

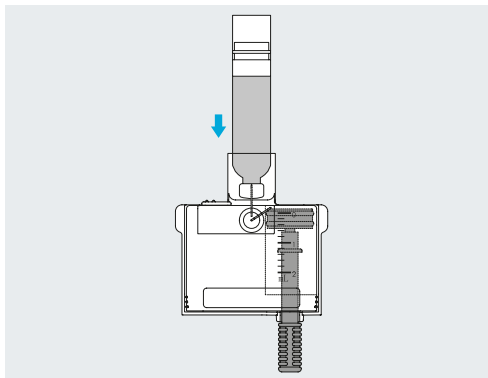


2 Navucite inzulin u spremnik.

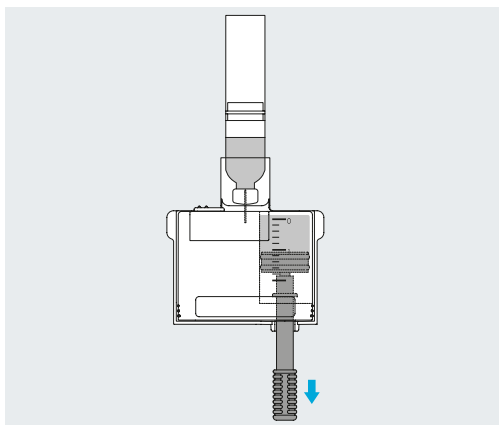


3 Uklonite mjehuriće zraka. Ponavljajte korake 1 do 3 dok spremnik ne bude napunjen.

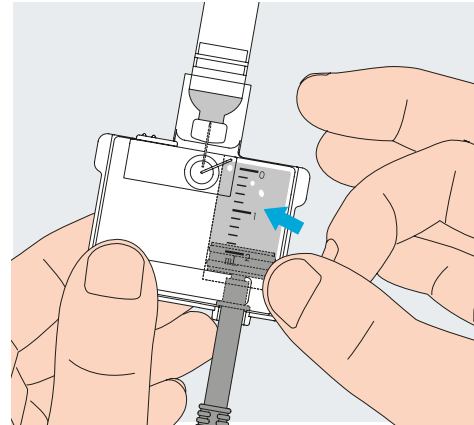


4 Uklonite bočicu.

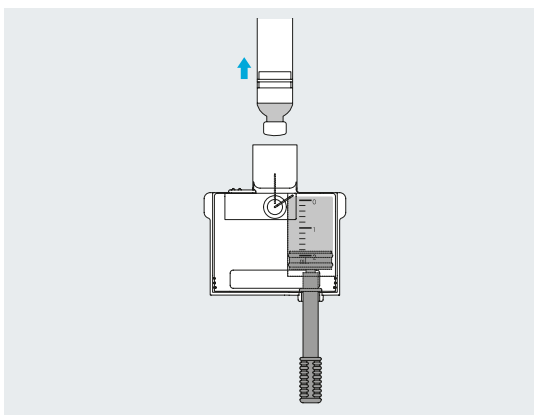

5 Uklonite adapter.

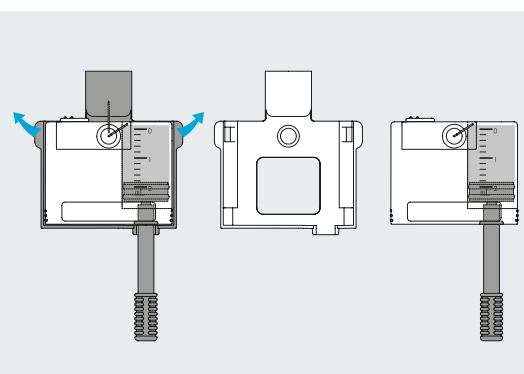


6 Uklonite potisnik klipa.

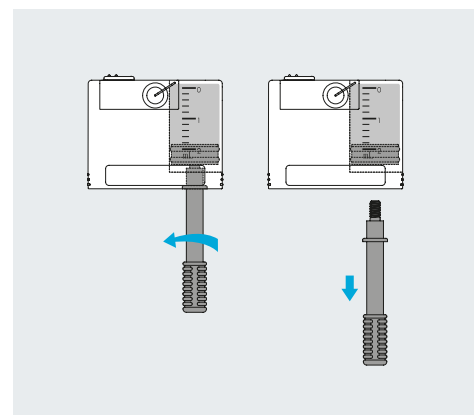


1C Pripremite novi infuzijski set


1 Operite ruke.

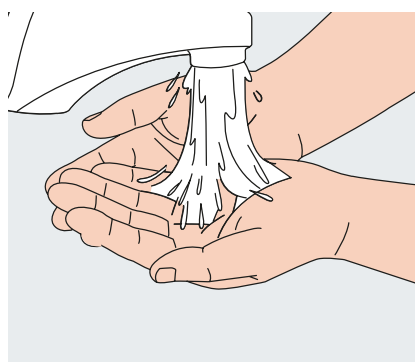


2 Odaberite mjesto, i pričvrstite bazu pumpe.

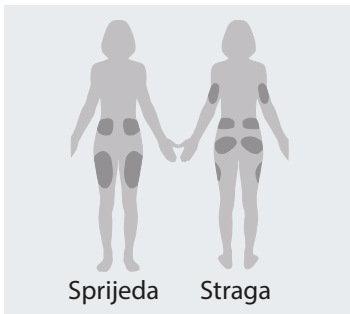


3 Otvorite ambalažu kanile i stavite ju u aplikator.

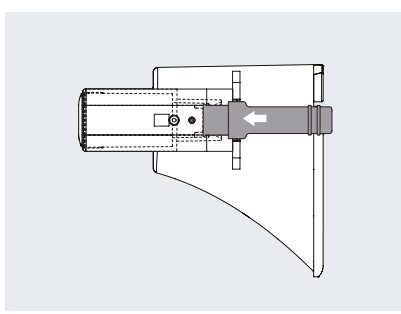


4 Skinite zaštitni poklopac.

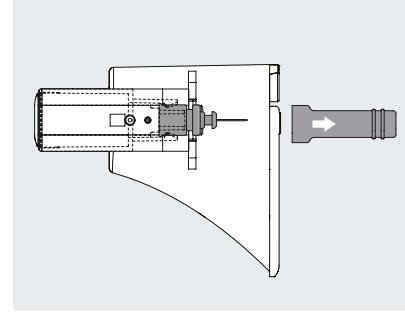


5 Postavite aplikator u bazu.

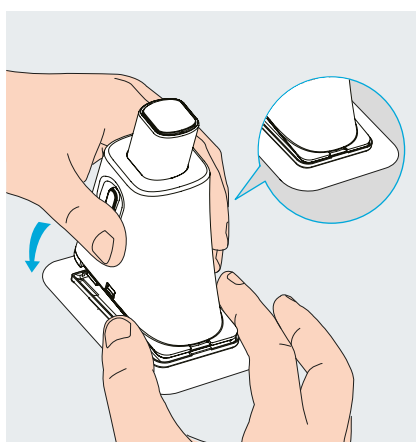

6 Pritisnite gumbе za umetanje kanile.

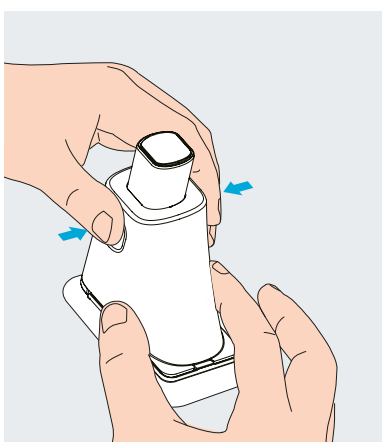

7 Uklonite aplikator s baze pumpe.

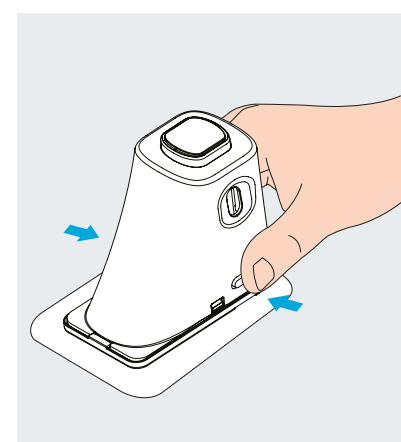

8 Zbrinite nosač igle

